

DETER
ILLICIT DRUG USE
IN THE HOSPITAL &
OUTPATIENT SETTING

COMBAT
THE OPIOID EPIDEMIC
MITIGATE
RISK
SAVE
LIVES



WANT TO LEARN MORE?
SCAN THE QR BELOW!



OUR STORY

Within a 4-month period in 2019, an affluent suburban hospital on the Intracoastal in Jacksonville Beach experienced two SIVAD overdose deaths on the bathroom floor of their Emergency Room.

One of these patients, a female in her 40s, was admitted to the ER for acute kidney injury and a reoccurring infection.


Despite being flagged as opioid dependent and being closely monitored, she was found unresponsive on the floor of an ER restroom after self-injecting opioids into her VAD and overdosing.

The doctor and nurse involved in her care took the deaths very personally, and searched the available market for a solution.

Their search found many industry professionals that were familiar or had experienced similar issues with patient tampering & self-injection, but little formal literature on the topic and few existing solutions that tried to address the problem.

They also surmised that if this problem has surfaced multiple times at a high-end hospital in an affluent, suburban area, it must also be prevalent across the US Healthcare system – especially in areas most affected by the Opioid Epidemic.

So the doctor and nurse practitioner, began working to develop their own solution – the IV CareLock.

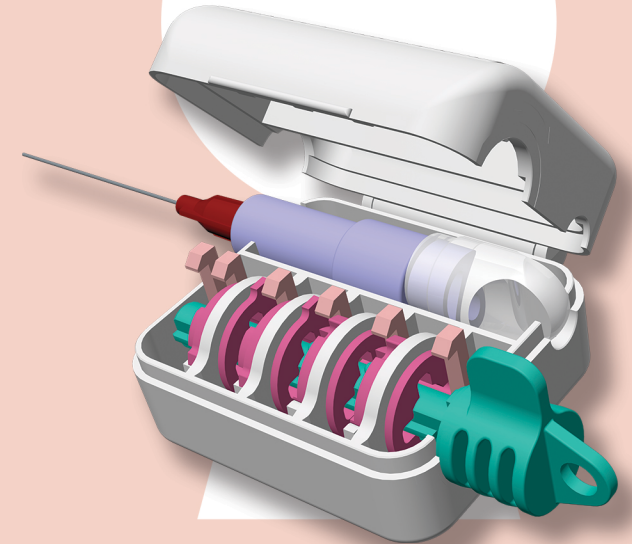

SECURE
INTRAVENOUS ACCESS SITES

PREVENT
PATIENT TAMPERING
& SELF INJECTION

PROVIDE
PEACE OF MIND
TO CAREGIVERS

PROTECT
YOUR MOST VULNERABLE PATIENTS

CONTACT US

IVCARELOCK.COM

INFO@IVCARELOCK.COM




INITIAL USE

STEP 1

The IV CareLock comes in the closed position.

STEP 2

Insert the key into the IV CareLock, with the key handle horizontal facing left.

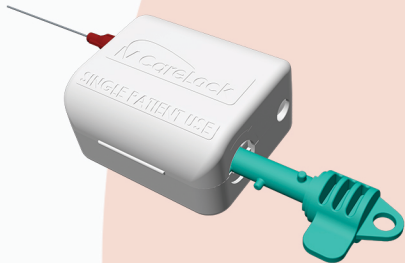

STEP 3

Rotate the key clockwise 90° to unlock and open the IV CareLock.


STEP 4

Rotate the key counterclockwise 90° to remove the key.

PLACEMENT

STEP 1

After the IV is placed, locate the pigtail catheter.

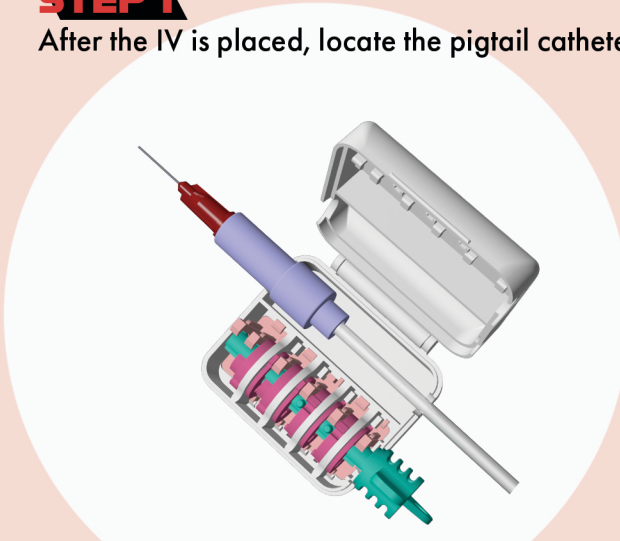

STEP 2

Position the pigtails IV hub with cap or attached tubing and close the cover over the base until it snaps into place.

STEP 3

If tubing with Y-ports is needed, place the device over any Y-port access points.

REMOVAL

STEP 1

Insert the key into the IV CareLock, with the key handle horizontal facing left.

STEP 2


Rotate the key clockwise 90° to unlock and open the IV CareLock. Remove the IV CareLock from the IV.

STEP 3

Rotate the key counterclockwise 90° to remove the key.

STEP 4

Dispose of the device with the Infusion Set in an appropriate manner.

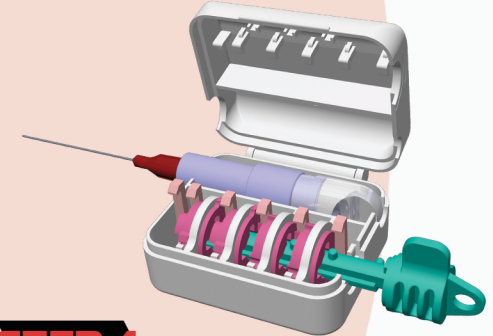

PROTECTION FOR EVERY ACCESS POINT ALONG AN INFUSION SET

- ✓ PIC LINES
- ✓ PIVV LINES
- ✓ CENTRAL LINES
- ✓ IV PIGGYBACKS

WATCH OUR HOW-TO VIDEO!

